



ACTIVARMOR

SUPPORT • ACTIVE • LIFESTYLE

WATERPROOF 3D PRINTED CASTS

PROVIDER GUIDE



ActivArmor is a custom plastic cast that is:

- Waterproof and breathable
- Durable yet lightweight
- Fixed or adjustable
- Convertable from locked-on cast to removable splint
- Made with holes to observe skin, wounds, & incisions
- Latex-free, hypoallergenic, & sanitizable



As Presented at:



And Proudly Partnered With Top Orthopedic Clinics Like:



Wrist
Cockup

Thumb
Spica


Hand
Based

Ulnar/Radial
Gutter

Functional
Splints

Non-Weight
Bearing AFO

Walking Boot

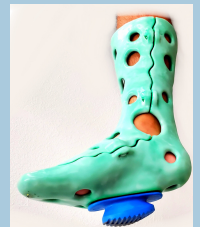

2 WAYS TO FIT PATIENTS FOR AN ACTIVARMOR CAST

Write a Prescription



1. Patient is put into temporary bivalved cast or splint until permanent ActivArmor cast arrives



2. Physician writes prescription and gives it to the patient (sample below)



3. Patient works with ActivArmor ortho tech to scan. Patient orders & pays for the cast.



4. ActivArmor ships finished cast to patient



5. ActivArmor ortho tech works with patient for any fittings or adjustments



6. Patient returns to office for follow-up wearing ActivArmor cast for approval

Questions?



Call Us Now
1-800-583-6690



Learn more at
activarmor.com/become-a-provider



ACTIVARMOR
SUPPORT • ACTIVE • LIFESTYLE

2828 Granada
Pueblo, CO 81005
800-583-6690
www.ActivArmor.com

Patient Name: _____

Date: _____

Rx

Diagnosis/CD10: _____
Precautions: _____

Order: Custom Orthosis

Orthotic Fabrication and Fitting (Check all that apply):

- Wrist Cook-up _____
- Forearm-Based Thumb Spica/MP Block/Ulnar/Radial Gutter
- Digits: I II III IV V Joints: MP PIP DIP
- Hand-Based Thumb Spica/MP Block/Ulnar/Radial Gutter
- Digits: I II III IV V Joints: MP PIP DIP
- Hand-Based Functional splint _____
- Long Arm orthosis _____
- Digits: I II III IV V Joints: MP PIP DIP
- Elbow orthosis _____
- Ankle foot orthosis – Non-Weight Bearing _____
- Ankle foot orthosis – Walking Boot _____

Thickness: Protective (Cast)

Thin/Lightweight (Splint, not for AFOs)

Strength Coating: Yes (recommended for athletes/durability/strength)

Closing Option: Locked-On Removable Transition Kit

Positioning/Custom Design Instructions: _____

Signature: _____ Date: _____

Physician Name: _____ NPI: _____

Or, Become a Partner Provider



1. Provider scans during in-office visit using the ActivArmor iPhone app



2. Provider returns patient to temporary bi-valved cast or splint



3. Cast is shipped to provider who fits & adjusts ActivArmor cast



4. Provider is reimbursed through insurance, Medicare, & Medicaid. (Self-pay option also available.)